

HARTMANN



BacilloI® 30 Sensitive Tissues

Material-friendly cleaning and disinfection wipes for sensitive surfaces



Characteristics

- 100 % plastic-free fabric material
- Low-lint, tear-resistant and free from microplastics
- High cleaning performance
- Tested under dirty conditions – cleaning and disinfecting in one step
- Very good skin compatibility – Dermatologically tested
- Can be used without gloves*

CE 0482

Areas of application

BacilloI® 30 Sensitive Green Tissues are low-alcohol, ready-to-use wipes for cleaning and disinfecting sensitive surfaces. Also suitable for displays, keyboards, mobile phones and tablets and sensitive surfaces – e.g. made of polycarbonate (PC), acrylic glass (PMMA) and polysulfone as well as artificial leather. It can also be used in commercial kitchens and food areas. The wipes soaked with BacilloI® 30 Sensitive Foam wet the surfaces excellently and act rapidly.

Microbiology

Bactericidal, yeasticidal, tuberculocidal mycobactericidal, virucidal against enveloped viruses (incl. HBV, HIV, HCV), limited spectrum virucidal activity

Proven efficacy	Testing method	Condition	Exposure time
Bacteria / Fungi			
Bactericidal / Yeasticidal	EN 16615 / EN 13727 / EN 13624	dirty	1 min
Tuberculocidal / Mycobactericidal	EN 14348	dirty	3 min
Viruses			
virucidal against enveloped viruses (incl. HBV, HIV, HCV)	EN 14476 / EN 16777	dirty	1 min
limited spectrum virucidal activity	EN 14476	dirty	30 min
Norovirus	EN 14476	dirty	2 min
Food / industry			
Bactericidal / Yeasticidal	EN 1276 / EN 1650 / EN 13697	dirty (20 °C)	1 min

* if infection and occupational safety permit.



Bacillo[®] 30 Sensitive Tissues



Directions for use

Wipe all surfaces. Ensure complete coverage. Reseal the package directly after use. Removal of residues is not needed. Surfaces directly in contact with food should be rinsed with water after application. When switching products, carry out intermediate cleaning.

Disinfectable area (per tissue)

Green Tissues:

- 120 Tissues (17 x 20 cm): 1.5 m²
- 80 Tissues XL (20 x 30 cm): 2 m²
- 160 Tissues (18 x 20 cm): 1 m²

PET Tissues:

- 80 Tissues (18 x 20 cm): 1.5 m²
- 40 Tissues XXL (25 x 38 cm): 2.5 m²

Do not treat food and feed directly. Treat only smooth (non-absorbent / non-porous) materials.

Notes: Ensure that disinfectant does not penetrate electronic equipment. Follow manufacturer's instructions for reprocessing. Do not mix with other products.

Stability

In open and closed flowpack: 3 months.

Hazardous Classification

Attention



H226: Flammable liquid and vapour



H319: Causes serious eye irritation

Composition

Active substances in the soaking solution: Ethanol 140 mg/g; Propan-2-ol 100 mg/g; Propan-1-ol 60 mg/g, Amines, N-C10-C16-alkyltrimethylenedi-, reaction products with chloroacetic acid (CAS number 139734-65-9) 2 mg/g.

Chemical-physical data

The information refers to the Bacillo[®] 30 Sensitive Foam soaking solution:

Flashpoint (DIN EN ISO 3679) 31 °C
 Density (20 °C) ca. 0,95 g/cm³
 PH Value (20 °C): ca. 8

Expert assistance

HARTMANN SCIENCE CENTER

Tel: +49 (0) 40 - 54 00 6 -111

E-Mail: science-center@hartmann.info



Research for infection protection
hartmann-science-center.com

Product name	Content	Item no
Bacillo[®] 30 Sensitive Green Tissues	120 tissues flow pack Size 17 x 20 cm	on request
Bacillo[®] 30 Sensitive Green Tissues Big Pack	160 tissues flow pack Size 18 x 20 cm	on request
Bacillo[®] 30 Sensitive Green Tissues XL	80 tissues flow pack Size 20 x 30 cm	on request
Flowpack Wallholder	1 Wallholder	on request
Flowpack-Holder flex	1 Holder	on request

Please note: That the availability of Bacillo[®] products may vary in different countries and regions. Contact your local distribution partner for more information.

The recommendations regarding our preparations are based on scientific tests and are given in good faith. More detailed recommendations, e.g. regarding material compatibility, are possible only in separate, individual cases. Our recommendations are not binding and do not constitute a guarantee. They do not preclude a company's own testing for the intended purpose and process. In this respect we cannot accept any liability. This is in accordance with our general conditions of sale and supply.

Use surface disinfectants safely. Always read the label and product information before use.



Helps. Cares. Protects.

PAUL HARTMANN AG
Paul-Hartmann-Straße 12
89522 Heidenheim
Germany

Phone: +49-7321-36-0
Fax: +49-7321-36-3636
info@hartmann.info
www.hartmann.info

Manufacturer: BODE Chemie GmbH, Melanchthonstraße 27, 22525 Hamburg
A company of the HARTMANN GROUP