



## Bacillo® Zero Tissues

Organic-acid based, extensively effective and material friendly cleaning and disinfecting wipes.



CE 0482

### Characteristics

- 100% plastic free nonwoven
- virucidal in 2 minutes
- free from alcohol, QAC, peroxide, perfume and dye
- > 98,5% natural origin ingredients
- patented formulation – organic acid complex
- no hazardous labeling – useable without gloves\*
- excellent material compatibility with all surfaces close to the patient

**Areas of application / Intended purpose**  
Cleaning disinfection of non-invasive medical devices. Cleaning disinfection of invasive medical devices, not as end point of processing.

Bacillo® Zero Tissues are suitable for the convenient, material-friendly rapid disinfection of:

**Surfaces according to Biocidal Products Regulation (BPR) such as:**

- displays and keyboards or touch pads of sensitive communication equipment, e.g. mobile phones, computers
- sensitive surfaces – e.g. made from Makrolon®, acrylic glass, polysulphone, and artificial leather – in healthcare facilities

Proven efficacy	Testing method	Condition	Exposure time
<b>Bacteria / Fungi</b>			
Bactericidal / Yeasticidal	EN 13727 / EN 13624 / EN 16615	dirty	2 Min.
Tuberculocidal / Mycobactericidal	EN 14348 / EN 16615	dirty	60 Min.
<b>Viruses</b>			
Virucidal	EN 14476 / EN 16615**	dirty	2 Min.

\* If infection control and occupational safety protocols permits it

\*\* Modified 4-field test


**Medical equipment and devices such as:**

- displays and keyboards or touch pads of portable and stationary medical equipment, e.g. monitoring systems
- sensitive, non-invasive medical devices, e.g. mammography equipment, operating theatre lamps

### Microbiology

Bactericidal, yeasticidal, tuberculocidal, mycobactericidal, virucidal activity against enveloped viruses (incl. HBV, HIV, HCV), limited spectrum virucidal activity, virucidal

# Bacillol® Zero Tissues

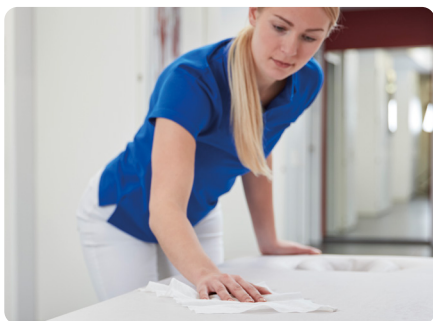

## Directions for use

Thoroughly wipe surfaces with Bacillol® Zero Tissues. Ensure complete coverage so that optimum disinfection is guaranteed. Thanks to their composition, Bacillol® Zero Tissues may also be used without gloves, if infection and occupational safety measures so permit. Dispose of used tissues. To prevent wipes from drying, reseal the package directly after use.

For use against mycobacteria: Use only on surfaces where they can be kept away from patients and staff for the duration of the exposure time.

It needs to be made sure that no disinfectant gets inside electronic devices. Carefully review and adhere to the instrument manufacturer's reprocessing instructions.

For single use.

When switching from one product to another, an intermediate cleaning has to be carried out.

## Stability

In open and closed flowpack: 3 months.

## Composition

(+)-tartaric acid in 5 mg/g and sodium benzoate in 5 mg/g

## Chemical-physical data

The information refers to the Bacillol® Zero soaking solution:

Flashpoint  
(DIN EN ISO 3679) non-flammable  
Density (20 °C) 1 g/cm<sup>3</sup>  
PH Value (20 °C): 2,1-2,4

## Expert assistance

HARTMANN SCIENCE CENTER

Tel: +49 (0) 40 - 54 00 6 -111

E-Mail: [science-center@hartmann.info](mailto:science-center@hartmann.info)



**Research for  
infection protection**  
[hartmann-science-center.com](http://hartmann-science-center.com)

Product name	Content	Item no
<b>Bacillol® Zero Tissues</b>	100 Tissues Flowpack Size 20 x 17 cm	on request
<b>Bacillol® Zero Tissues Big Pack</b>	160 Tissues Flowpack Size 20 x 18 cm	on request
<b>Bacillol® Zero Tissues XL</b>	80 Tissues Flowpack Size 20 x 30 cm	on request
<b>Flowpack Wallholder</b>	1 Wallholder	on request
<b>Flowpack-Holder flex</b>	1 Holder	on request

Please note: That the availability of Bacillol® products may vary in different countries and regions. Contact your local distribution partner for more information. The recommendations regarding our preparations are based on scientific tests and are given in good faith. More detailed recommendations, e.g. regarding material compatibility, are possible only in separate, individual cases. Our recommendations are not binding and do not constitute a guarantee. They do not preclude a company's own testing for the intended purpose and process. In this respect we cannot accept any liability. This is in accordance with our general conditions of sale and supply.

*Use surface disinfectants safely. Always read the label and product information before use.*



**Helps. Cares. Protects.**

Manufacturer:

BODE Chemie GmbH

A company of the HARTMANN GROUP

Tel: +49 40 5 40 06-0 · Fax: -200

[info@bode-chemie.de](mailto:info@bode-chemie.de)

[www.bode-chemie.com](http://www.bode-chemie.com)